




BARD® Intermittent Catheters (Latex)

Instructions For Use

Indications For Use: For urological use only.

-  Single-use. Single patient use, do not reuse.  Do not use if package is opened or damaged.

Storage: Store catheters at room temperature away from direct exposure to light, preferably in the original box.

 **Caution:** This product contains natural rubber latex which may cause allergic reactions.

Warnings:

- On latex and red rubber catheters, do not use ointments or lubricants having a petrolatum base. They will damage latex.
- Visually inspect the product for any imperfections or surface deterioration prior to use.
- Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to failure, and/or to injury, illness or death of the patient.




Note: After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Adverse Reactions

Urinary Tract Infection
Bleeding from the urethra
Irritation of the urethra

Instructions for Intermittent Catheters:

1. Wash your hands thoroughly with soap and water.
2. Remove catheter from the pack.
3. Position yourself comfortably, cleaning the opening of the urethra and surrounding area.
4. Gently insert rounded end of catheter into urethra.
5. When urine stops flowing, remove catheter from urethra.
6. Dispose of catheter in accordance with local rules and regulations.
7. Wash your hands.

 **Caution:** Federal (U.S.A.) laws restricts this device to sale by or on the order of a physician.

BARD



Manufacturer:
C. R. Bard, Inc.
Covington, GA 30014
1-800-526-4455
www.bardmedical.com
Assembled in Mexico



Bard, Bardex and Bardia are trademarks and/or registered trademarks of C. R. Bard, Inc.
© 2015. C. R. Bard, Inc.

PK7639306 06/2015