BARD® TOUCHLESS® Plus Unisex Pre-Lubricated Urethral Catheter Kit

**Single Use Only
Sterile**

Contents of kit are sterile unless package is opened or damaged.

Contents:
Vinyl Urethral Catheter
1100cc collection bag

Caution:

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

LATEX-FREE

BARD

C. R. Bard, Inc.
Covington, GA 30014 USA
1-800-526-4455
www.bardmedical.com

Warning: This is a single use device. Do not re-sterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Instructions for Use:

Preparing for Catheterization:

1. Open kit and remove contents.

Male/Female Catheterization:

1. Remove cap from insertion tip while squeezing tabs. Save cap for closing the bag.
2. Slide the catheter halfway into the insertion tip.
3. **Male** - Prepare the male urethral meatus and the surrounding area with povidone-iodine swabs.
Female - Prepare the female urethral meatus by holding the outer labia apart and prep the urethral meatus and surrounding area with povidone-iodine swabs.
4. **Male** - Holding the penis, advance the insertion tip into the urethra no further than the flange base.
Female - Spread the inner labia, advance the insertion tip into the urethra no further than the flange base. Release the inner labia.

5. Place your nondominant hand on the finger control guide to stabilize catheter in urethra. With your dominant hand, grasp catheter through bag approximately 1" below the finger control guide and push catheter into urethra.
The catheter should be introduced by short, repetitive pushing motions. Repeat motions until catheter reaches bladder and urine starts to flow.

6. Allow urine to flow freely, making certain the catheter guide is elevated at least 4" above lower portion of the bag. Allow flow until bladder is empty or until bag is filled.

PRECAUTION: It is recommended that the collection bag be held. The catheter could possibly separate from the collection bag when urine increases weight of the bag.


7. Withdraw the catheter from the urethra. Remove the remaining portion of the catheter from the collection bag by pulling it through the insertion tip. **Note: The catheter is designed to pass through the insertion tip.**
8. The filled collection bag may be closed by replacing the cap over the insertion tip. Make sure the cap snaps on securely.

Specimen Collection:

To collect a specimen, obtain a sample cup/tube and alcohol wipe. Remove guide tip by pulling tab(s) upward from bag. Wipe port with alcohol. Pour specimen through port at top of bag into cup/tube.

Draining Bag:

Remove insertion tip by pulling tab(s) upward from bag. Drain urine through port at top of bag.

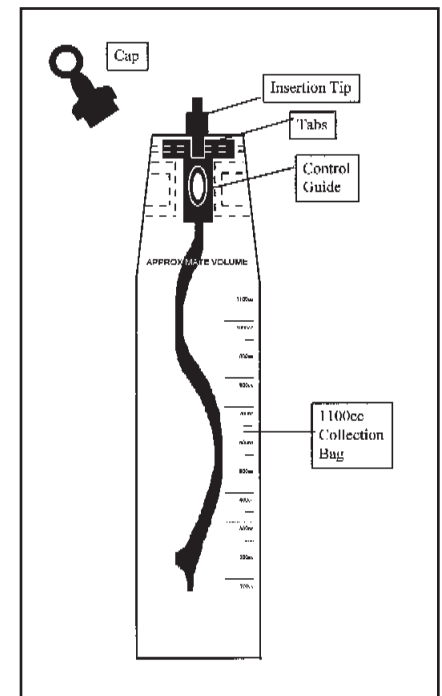Manufactured in Mexico.
Bard and Touchless are trademarks and/or registered trademarks of C. R. Bard, Inc.

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Bard y Touchless son marcas comerciales y/o comerciales registradas de C. R. Bard, Inc.