

Touchless® Plus

INTERMITTENT CATHETER SYSTEM



Leading the way to independence

BAIRD

CARE

Care. Comfort. Choice.™

Safe, Sterile and Simple

The BARD® TOUCHLESS® Plus Intermittent Catheter System

The BARD® TOUCHLESS® Plus Intermittent Catheter system has what is known as a “closed system” designed to help reduce the risk of urinary tract infections. Among the features that you’ll find helpful are:

- 1 Insertion tip** correctly positions the catheter in the urethra to help reduce UTIs.
- 2 Catheter control guide** allows control of catheter insertion with the touch of a finger.
- 3 Tapered neck design** decreases cuts and scrapes associated with sharp corners.
- 4 Catheter cap** has pull ring for easy removal.
- 5 Pre-lubricated catheter** reduces discomfort during insertion.
- 6 Large 1100cc** collection bag is durable, virtually eliminating the risk of leaks.



Ease of Use

Easy to insert, easy to use, easy to remove – the features needed to enhance convenience and usage.

- Catheter cap has pull ring for easy removal – especially for people with limited dexterity.
- Pre-lubricated catheter reduces discomfort during insertion.
- Large 1100cc collection bag is durable, virtually eliminating the risk of leaks.



Safety

The BARD® TOUCHLESS® Plus Intermittent Catheter System is a closed system, designed to help reduce the risk of urinary tract infections.

- Insertion tip positions the catheter in the urethra, bypassing the meatus to help reduce UTIs.
- Catheter guide allows control of catheter insertion with the touch of a finger.
- Tapered neck design decreases excoriation associated with sharp corners.



Sterile, Easy to Use:

1. Remove cap 1 from insertion tip using pull ring.
2. Slide the pre-lubricated catheter 2 halfway into the insertion tip 3 and insert into urethra.
3. Holding the patented guide 4 with non-dominant hand, use dominant hand to grasp catheter through bag 5 and push catheter into urethra.
4. Once bag is filled, withdraw catheter through insertion tip, snap on cap and dispose.

Your System, Your Choice

- Choice of red rubber or vinyl catheters based upon your preference and needs.
- Variety of catheter sizes, and available with or without accessories.



Product Number	Description	French Size	Not made with natural rubber latex	HCPC Code
Accessories included - 3 PVI swabs, 2 gloves not made with natural rubber latex, underpad				
4A5108	Vinyl	8 Fr.	Yes	A4353
4A5110	Vinyl	10 Fr.	Yes	A4353
4A5142	Vinyl	12 Fr.	Yes	A4353
4A5144	Vinyl	14 Fr.	Yes	A4353
4A5146	Vinyl	16 Fr.	Yes	A4353
4A7114	Vinyl Coude	14 Fr.	Yes	A4353
4A7116	Vinyl Coude	16 Fr.	Yes	A4353
4A5042	Red Rubber	12 Fr.	No	A4353
4A5044	Red Rubber	14 Fr.	No	A4353
4A7044	Red Rubber Coude	14 Fr.	No	A4353
Accessories not included (catheter and collection bag only)				
4A6110	Vinyl	10 Fr.	Yes	A4353
4A6142	Vinyl	12 Fr.	Yes	A4353
4A6144	Vinyl	14 Fr.	Yes	A4353
4A6146	Vinyl	16 Fr.	Yes	A4353
4A8114	Vinyl Coude	14 Fr.	Yes	A4353
4A8116	Vinyl Coude	16 Fr.	Yes	A4353
4A6044	Red Rubber	14 Fr.	No	A4353

NOTE: Reimbursement policies vary widely from insurer to insurer and reimbursement policies of the same insurer may vary in different sections of the U.S. As reimbursement policies are subject to change, the information contained herein may not be accurate at the time of use. Prior to submission of a claim for reimbursement, the user should contact the insurer (i.e., Medicare, Medicaid, or private payer) to verify applicable codes and reimbursement levels. C. R. Bard, Inc. does not guarantee that the purchase of any similar item is reimbursement in whole or in part, by any public or private health care insurer.

For more information, please contact us at 1-800-243-3315 or visit www.bardcare.com.



Intermittent catheters are intended for draining urine from the bladder. Reuse of a single use device may create a risk of patient or user infection. Please consult product labels and inserts for more detailed safety information and instructions for use.

C.R. Bard, Inc.
Covington, GA 30014

Bard, Care, Comfort, Choice, and Touchless are trademarks and/or registered trademarks of C.R. Bard, Inc. ©2017 C.R. Bard, Inc. All Rights Reserved. Printed in U.S.A.
1702-23 BMD/TCHL/1017/0011