BARD

UltraFlex™

SELF-ADHERING MALE EXTERNAL CATHETER

Spirit® Style 1

HYDROCOLLOID ADHESIVE SHEATH

(EN)

DESCRIPTION/INDICATION

The self-adhering male external catheter is designed for the management of male urinary incontinence.

CONTRAINDICATION

Do not use on irritated or compromised skin.

PRECAUTION

Do not use if allergic reaction occurs. For good hygiene, change catheter daily. Use of a single device for longer periods than 24 hours may increase the risk of complications.

DIRECTIONS: TO APPLY

1) Wash penis with mild soap and warm water. Dry thoroughly.

2) Trim pubic hair if necessary.

3) Unroll self-adhering catheter over penis.

4) Gently squeeze the catheter to properly seal adhesive to the skin.

IMPORTANT: Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

5) Connect to drainage device.

DIRECTIONS: TO REMOVE

Gently roll catheter off the penis.

Note: If necessary, apply a warm wet compress (such as a wet washcloth) around the catheter to help loosen the adhesive.

REF
Catalog Number

LOT
Lot Number

		
Manufacturer	Authorized Representative	Units

			
Non-Sterile	Single Use	Use By	Not made with natural rubber latex

Manufacturer: C. R. Bard, Inc. Covington, GA 30014 USA (800) 526-4455 www.bardcare.com

EC REP Bard Limited, Forest House Crawley, West Sussex UK RH11 9BP +44 1293 527 888

Bard, Spirit and UltraFlex are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2016 C. R. Bard, Inc. All Rights Reserved. PK245030 01/2017

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